

## FLUIDOTHERAPY DIGITAL MODEL CALIBRATION

1. Unplug the unit from the power outlet.
2. While pressing and holding the start and stop buttons simultaneously, plug the unit back into the power outlet.
3. CAL will appear on the display, release the start and stop buttons.
4. Press the start button. The motor(s) will start and the designation 'LO' plus two digits will appear. Use the increase or decrease button to set the desired low end motor speed. LO16 is recommended.
5. Press the mode button, the motor(s) will stop and the designation 'HI' plus two digits will appear. Press start and the motors will start, use the increase or decrease buttons to set the desired high end motor speed. HI40 is recommended.
6. Press the mode button to save the motor speed settings and start the speed calibration.
7. The unit will now go through a 30-45 second motor speed calibration.
8. After the self calibration is completed, the designation 'PERF' will appear. If an error code should appear the complete procedure must be repeated.
9. To finish the calibration, press the 'mode' button until 'CAL' appears in the display.
10. Unplug the unit from the power outlet, wait 6 seconds and plug the unit back in.
11. You are now finished with the Calibration.
12. Call ERS BioMedical if you have further questions 800-479-2987

