



900 Black Box Wiring:

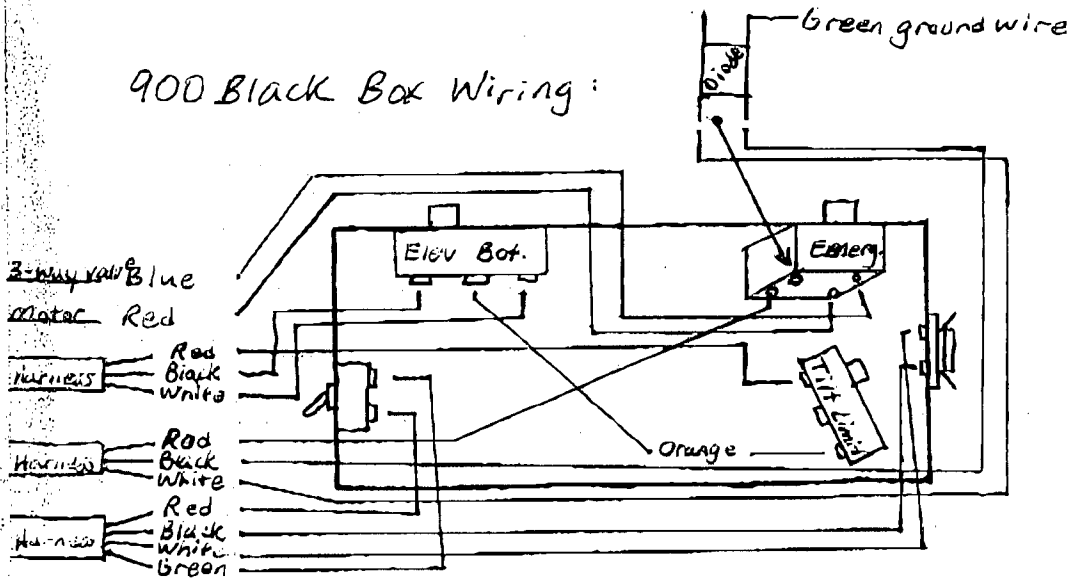


Z-FRAME WIRES

- GREEN TO GREEN
- PURPLE TO BLACK
- YELLOW TO GREEN
- WHITE TO WHITE
- RED TO RED
- BLACK TO BLACK
- ORANGE TO ORANGE
- BROWN TO BROWN
- BLUE TO WHITE

900 Z-Frame Connections:

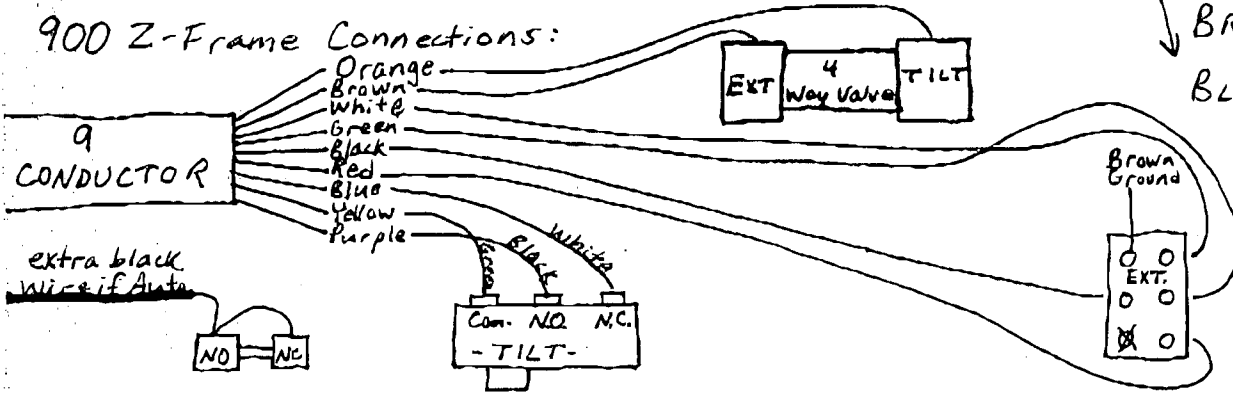


New style tilt limit switch:

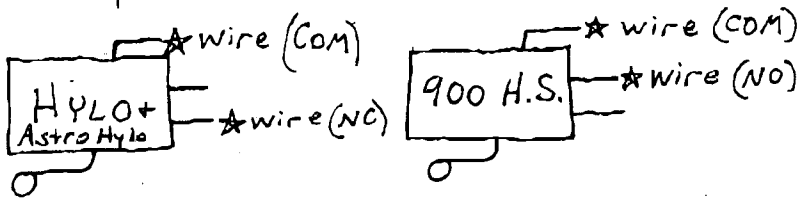

HYLO ENDCAP SWITCH WIRE TO # 3+4 in CPC HOUSING

McMANIS ELEV + 900 H.S. VOL CHECK AT BACK OF ELECTRICAL UN  
5 volt - BLACK + WHITE  
24 volt - BLACK + PURPLE

m+w motor (NO 4-way valve under pelvic)  
900 Z-frame connections

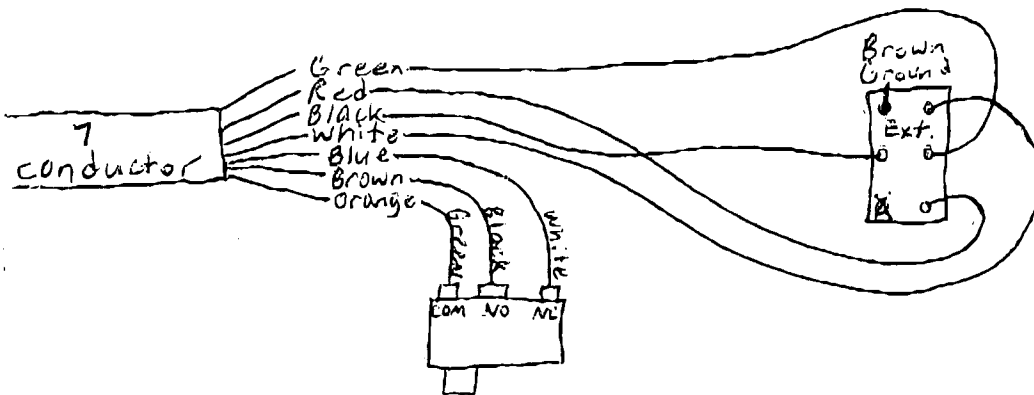

Old McManis El.  
Flex down valve - B  
Flex up valve - W

