

1

2

3

4

D

D

C


C

B

B

A

A

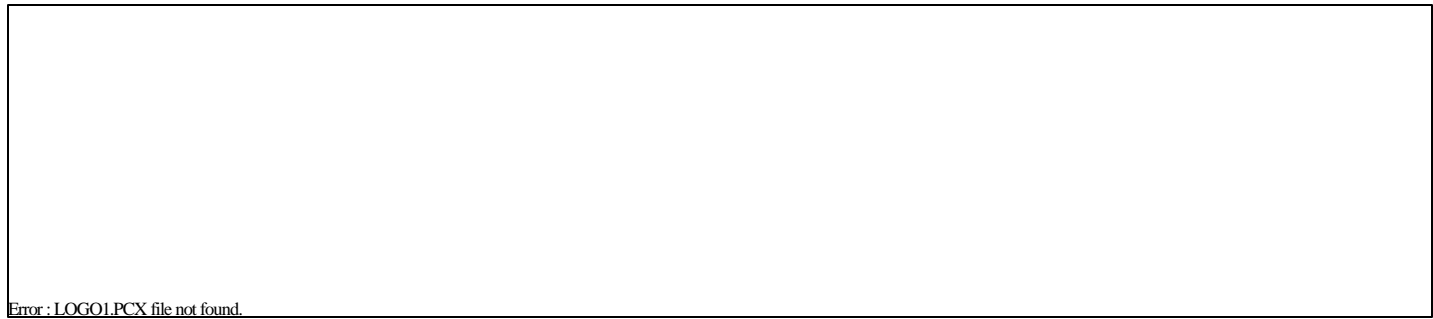

TheraMini VII Stimulator

Section I. PROCESSOR & USER INTERFACE
Section II. WAVEFORM GENERATION
Section III. POWER SUPPLY


SECTION I
section1.sch



SECTION II
section2.sch



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SECTION III
section3.sch



Title				<i>TheraMini Stimulator</i>				RICH-MAR CORP.			
Size:	A	Number:	00000000	Rev:	C	Sheet	1 of 15				
Date:	11-Jan-2002	Drawn by:	Bill Reddy								
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINI.DDB - Documents\TMINI.PRJ											

1

2

3

4

1

2

3

4

D

D

C

C

B

B

A


A

Section I. Processor & User Interface



- CPU and Memory (HC11/PSD312)
 - CPU Support, Serial I/O, Audible Output
- Front Panel I/O
 - Membrane Buttons, Encoder Input
 - Character & Graphic LCDs

CPU
cpu.sch

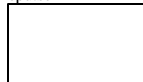


Title				<i>Section I. CPU & User I/F</i>		RICH-MAR CORP.	
Size: A	Number: 00000000	Rev: C	Sheet 2 of 15				
Date: 11-Jan-2002	Drawn by: Bill Reddy						
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINLDDB - Documents\section1.sch							

1

2

3

4

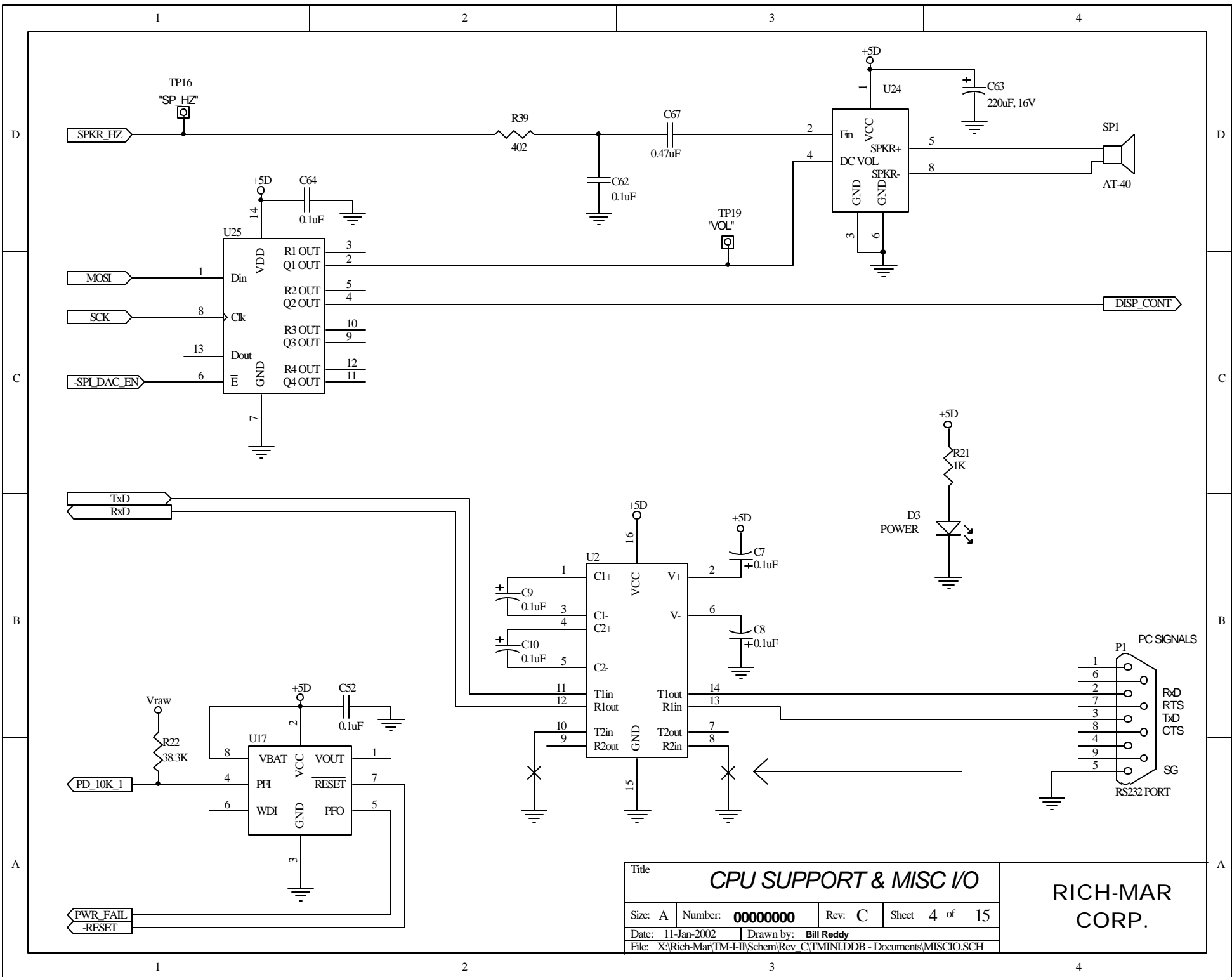

Title			
CPU SUPPORT & MISC I/O			
Size: A	Number: 00000000	Rev: C	Sheet 4 of 15
Date: 11-Jan-2002	Drawn by: Bill Reddy		
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINLDDB - Documents\MISCIO.SCH			

**RICH-MAR
CORP.**

1

2

3

4

D

D

C

C

B

B

A

A

Front Panel Inputs



- Encoder Knob with Push Buttton
- Membrane Switches

- Display Options:
 - 2 x 16 LCD Character Module
 - 128 x 64 Graphic LCD

MEMBRANE SWITCHES
BUTTONS.SCH



LCD DISPLAYS
DISPLAYS.SCH




Title				<i>FRONT PANEL I/O</i>		RICH-MAR CORP.	
Size: A	Number: 00000000	Rev: C	Sheet	5 of	15		
Date: 11-Jan-2002	Drawn by: Bill Reddy		File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINL.DDB - Documents\PANEL.SCH				

1

2


3

4

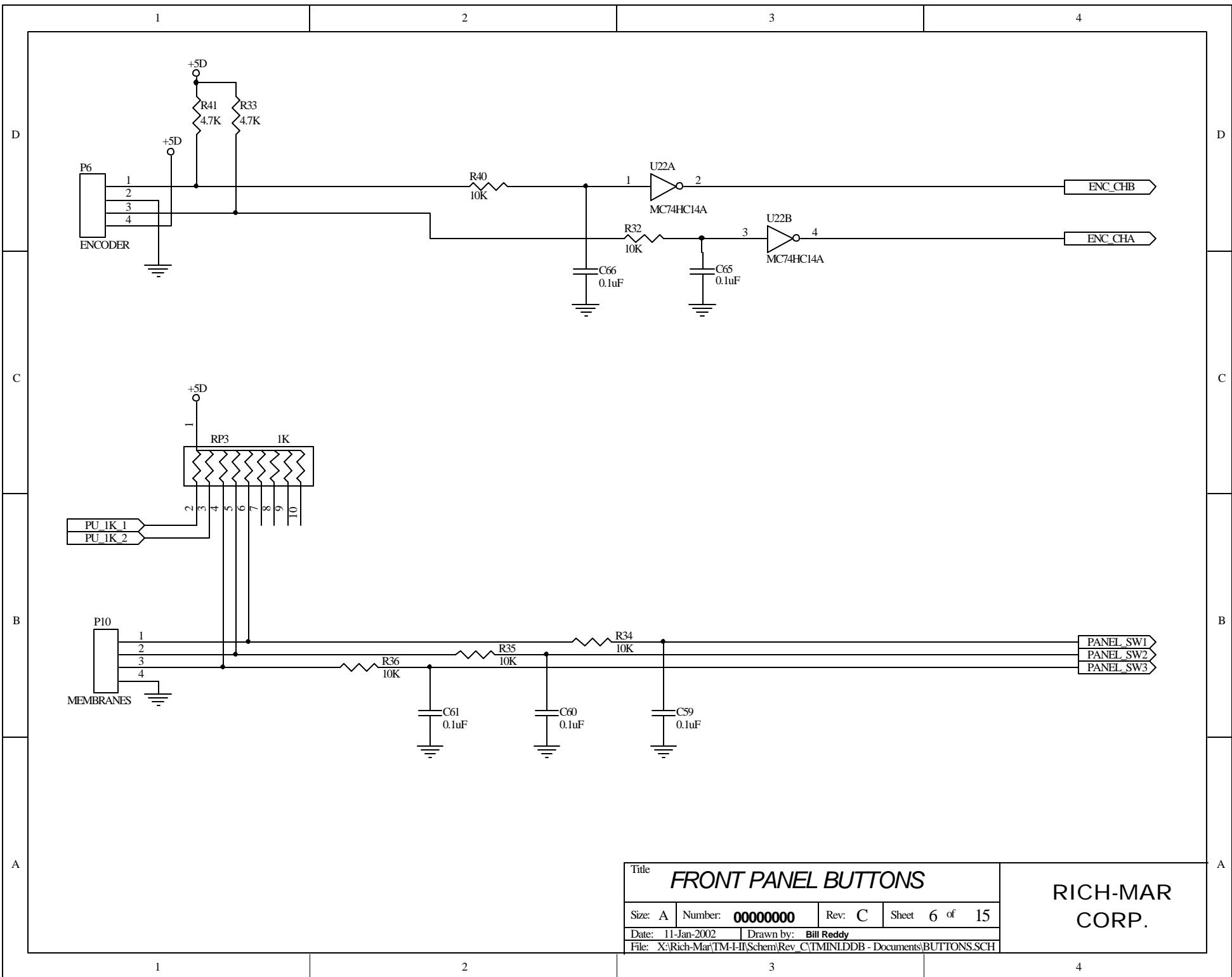

Title			
FRONT PANEL BUTTONS			
Size: A	Number: 00000000	Rev: C	Sheet 6 of 15
Date: 11-Jan-2002	Drawn by: Bill Reddy		
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINLDDB - Documents\BUTTONS.SCH			

**RICH-MAR
CORP.**

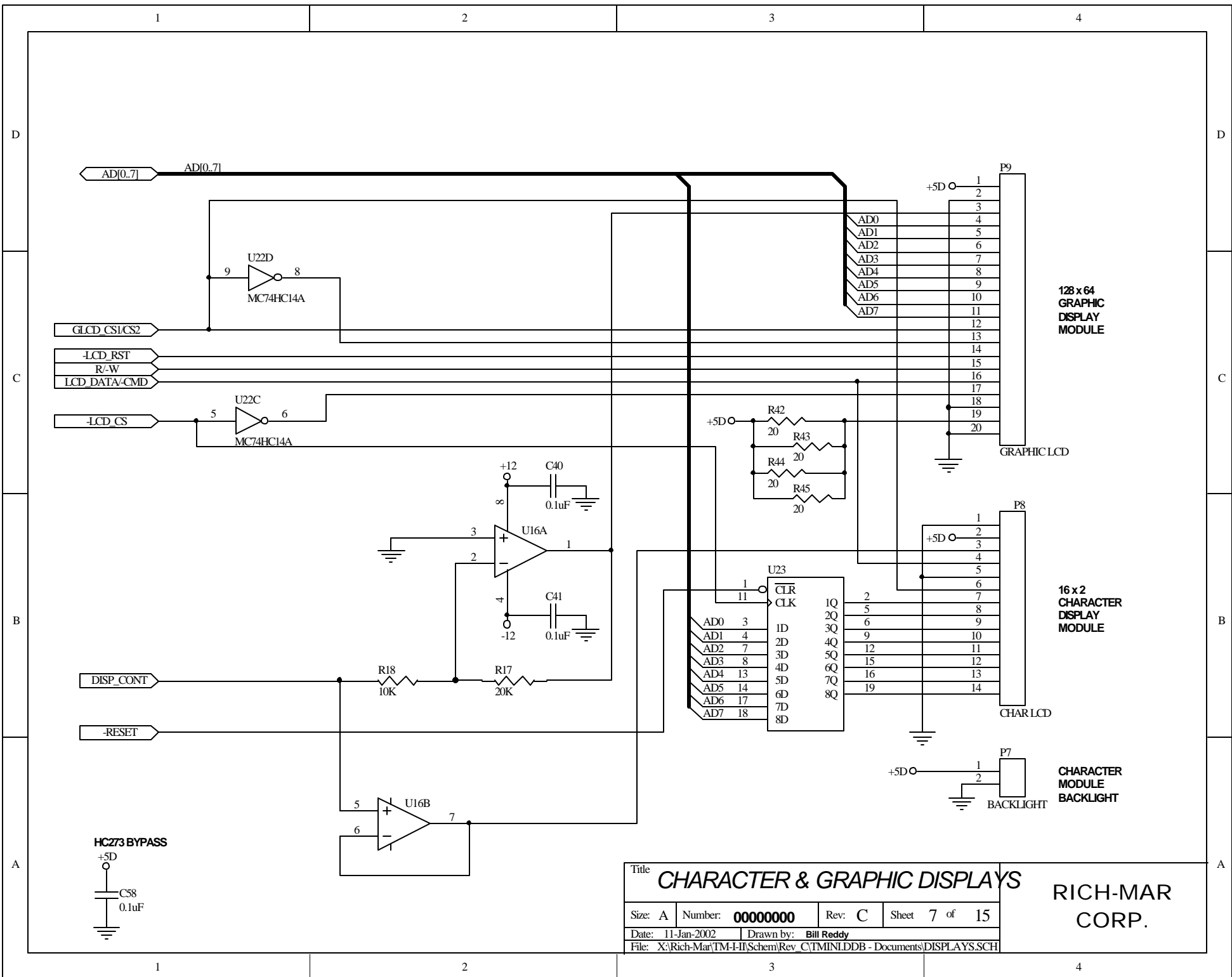

Title **CHARACTER & GRAPHIC DISPLAYS**

Size: A	Number: 00000000	Rev: C	Sheet 7 of 15
Date: 11-Jan-2002	Drawn by: Bill Reddy		
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINL.DDB - Documents\DISPLAYS.SCH			

**RICH-MAR
CORP.**

1

2

3

4

D

D

C

C

B

B

A

A

Section II. Waveform Generation

- Digital Signal Processor (ADSP2105)
 - Digital-to-Analog Converters
 - Filters
 - Amplifiers
 - Isolation Transformers & Output Monitoring

DSP
DSP.sch




Title				<i>Section II. Waveform Generation</i>		RICH-MAR CORP.	
Size: A	Number: 00000000	Rev: C	Sheet 8 of 15				
Date: 11-Jan-2002	Drawn by: Bill Reddy						
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINLDDB - Documents\section2.sch							

1

2

3


4

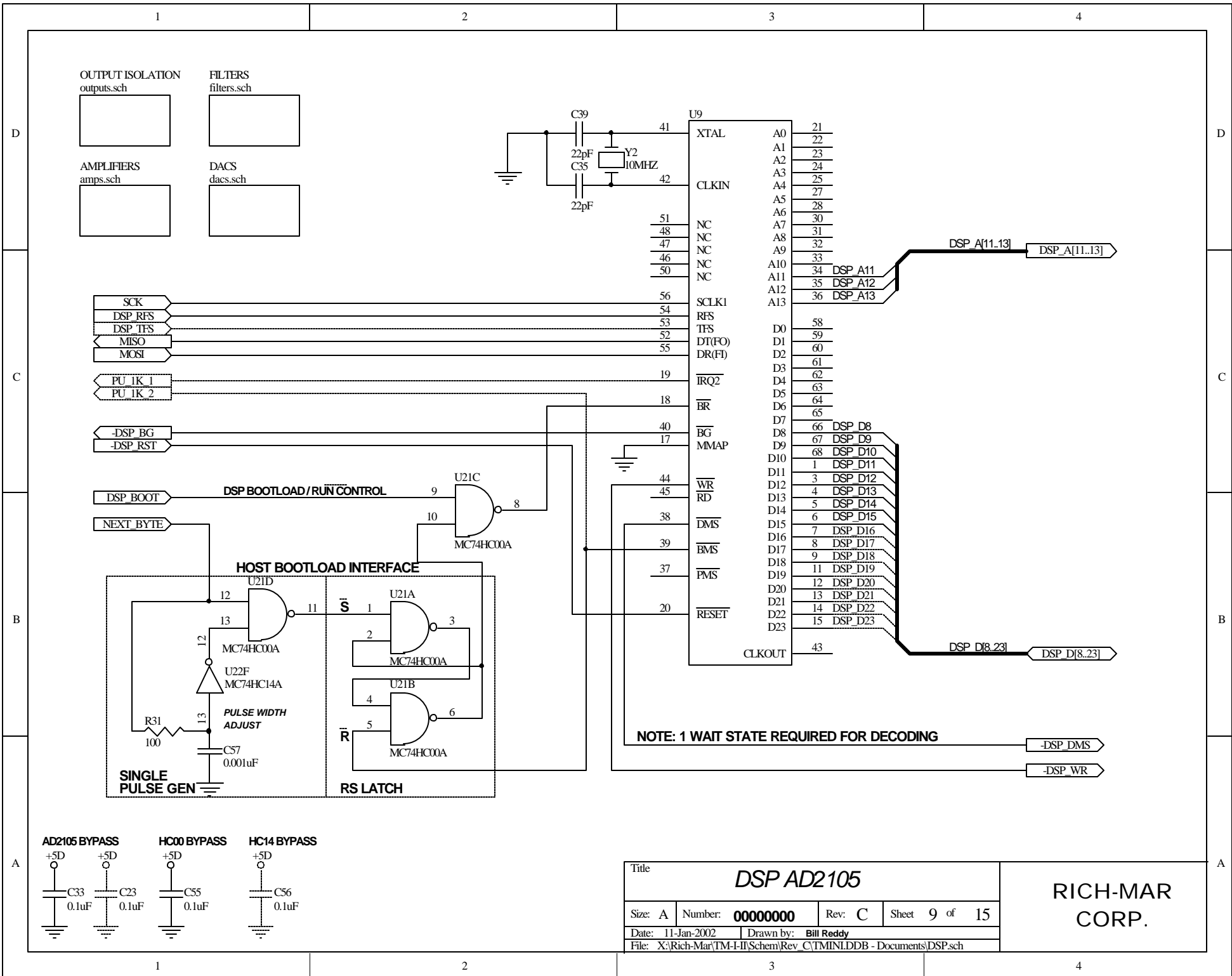

NOTE: 1 WAIT STATE REQUIRED FOR DECODING


-DSP_DMS

-DSP_WR

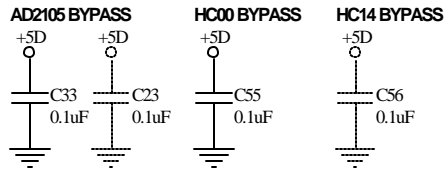


Title			
DSP AD2105			
Size: A	Number: 00000000	Rev: C	Sheet 9 of 15
Date: 11-Jan-2002	Drawn by: Bill Reddy		
File: X:\Rich-Mar\TM-I-IT\Schem\Rev_C\TMINI.DDB - Documents\DSP.sch			

**RICH-MAR
CORP.**

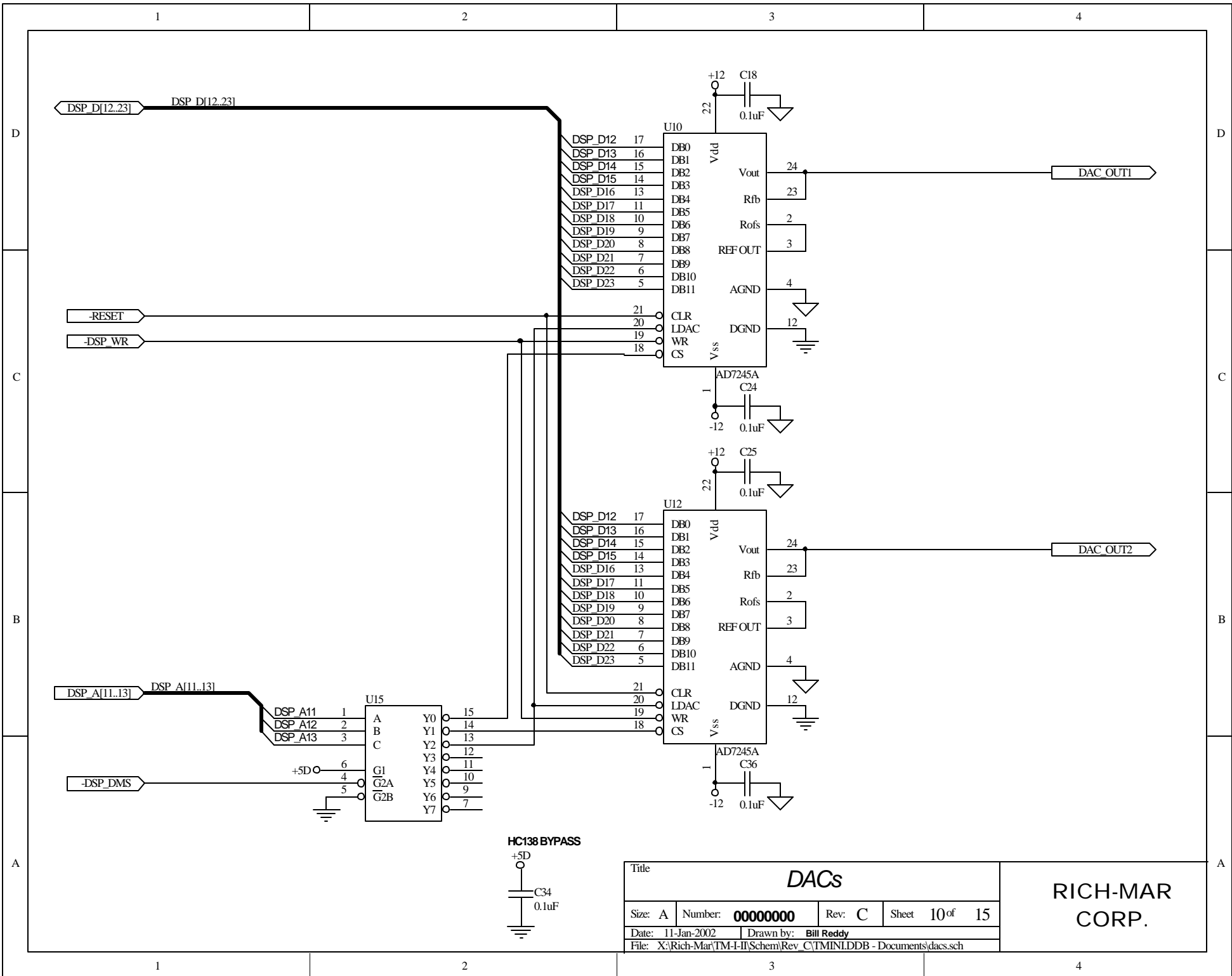


Title			
DACs			
Size: A	Number: 00000000	Rev: C	Sheet 10 of 15
Date: 11-Jan-2002	Drawn by: Bill Reddy		
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINLDDB - Documents\dacs.sch			

**RICH-MAR
CORP.**

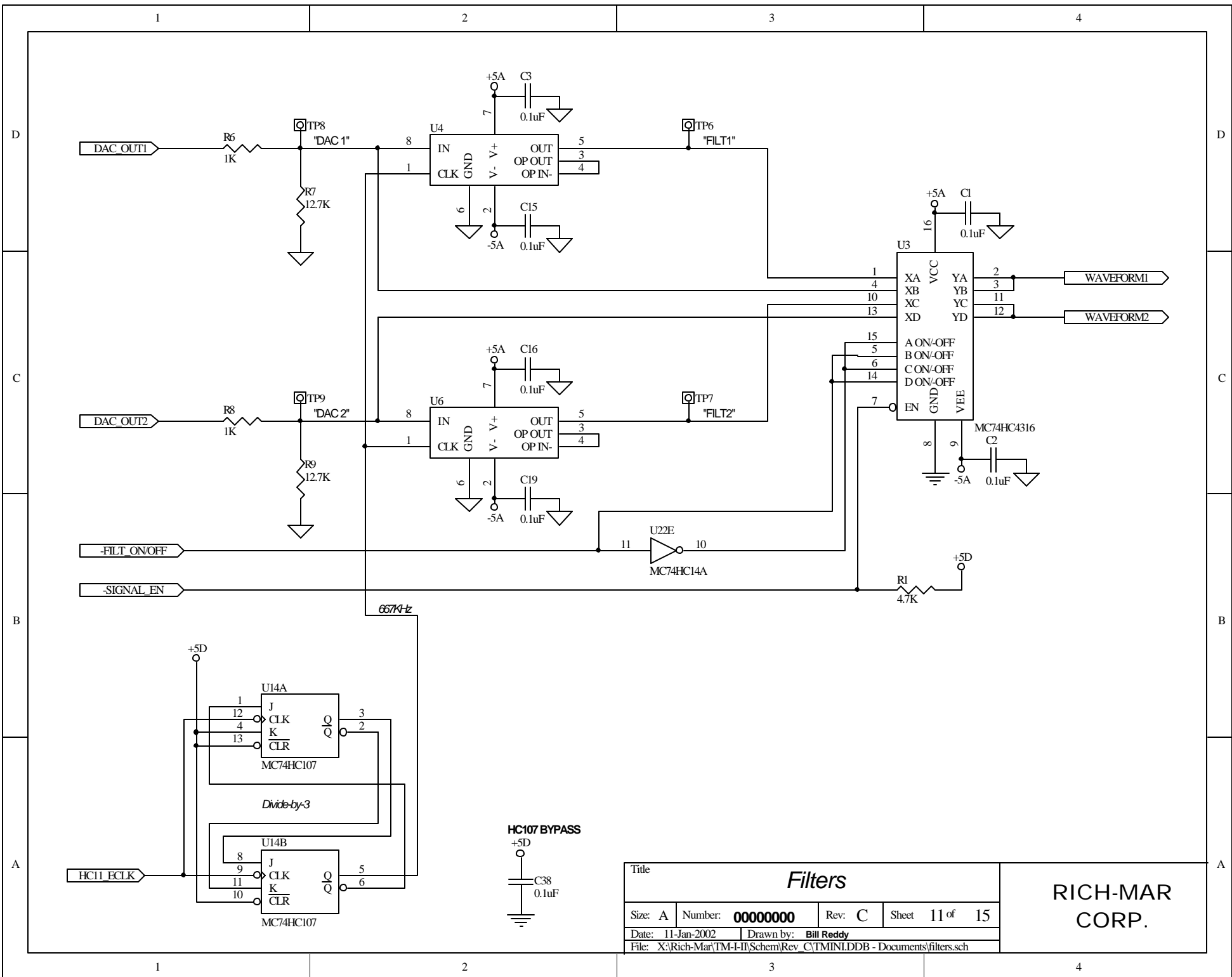


Title				Filters	
Size: A	Number: 00000000	Rev: C	Sheet 11 of 15		
Date: 11-Jan-2002		Drawn by: Bill Reddy			
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINLDDB - Documents\filters.sch					

**RICH-MAR
CORP.**

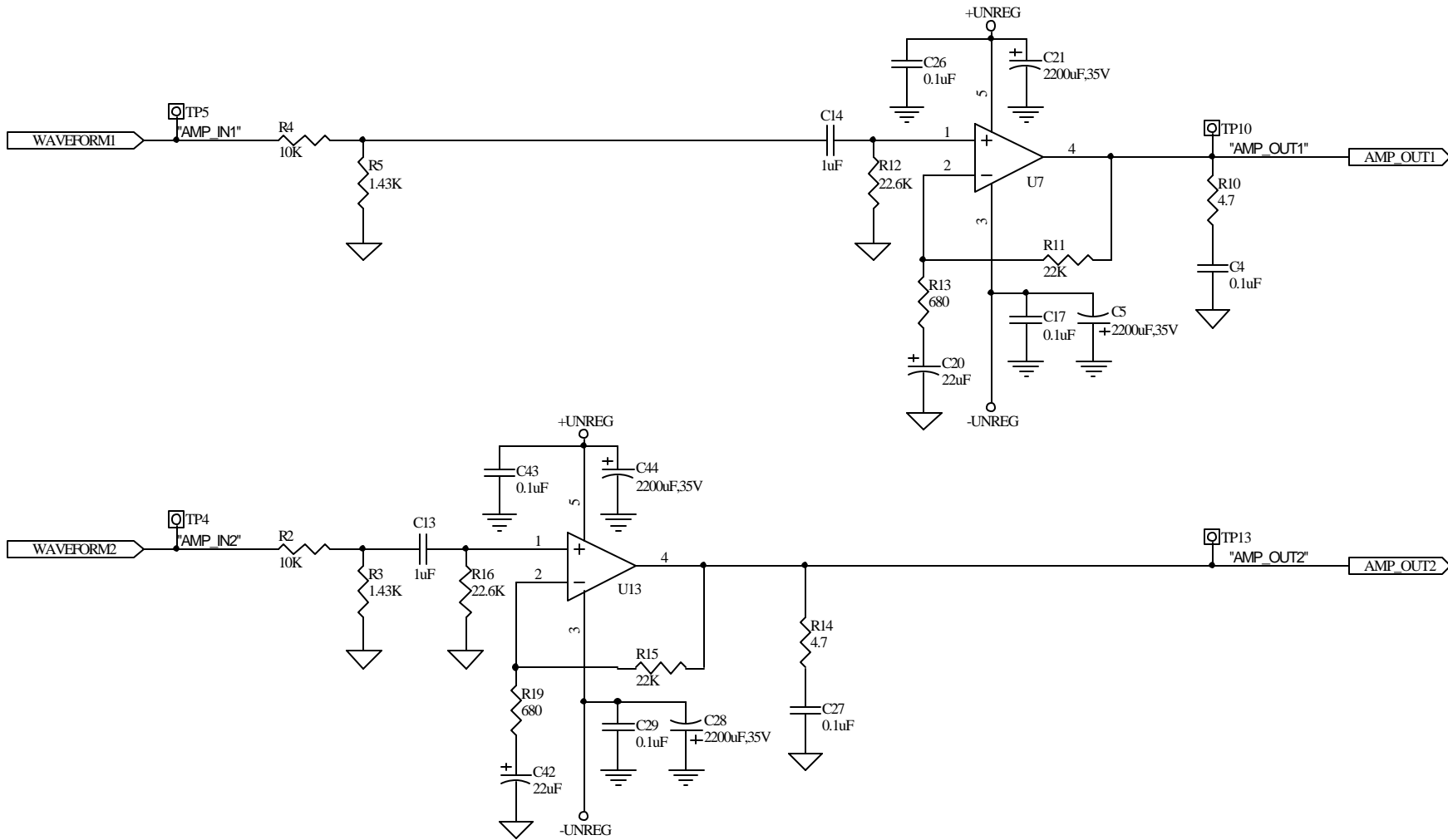

Title			
Output Amplifiers			
Size: A	Number: 00000000	Rev: C	Sheet 12 of 15
Date: 11-Jan-2002	Drawn by: Bill Reddy		
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINLDDB - Documents\amps.sch			

**RICH-MAR
CORP.**

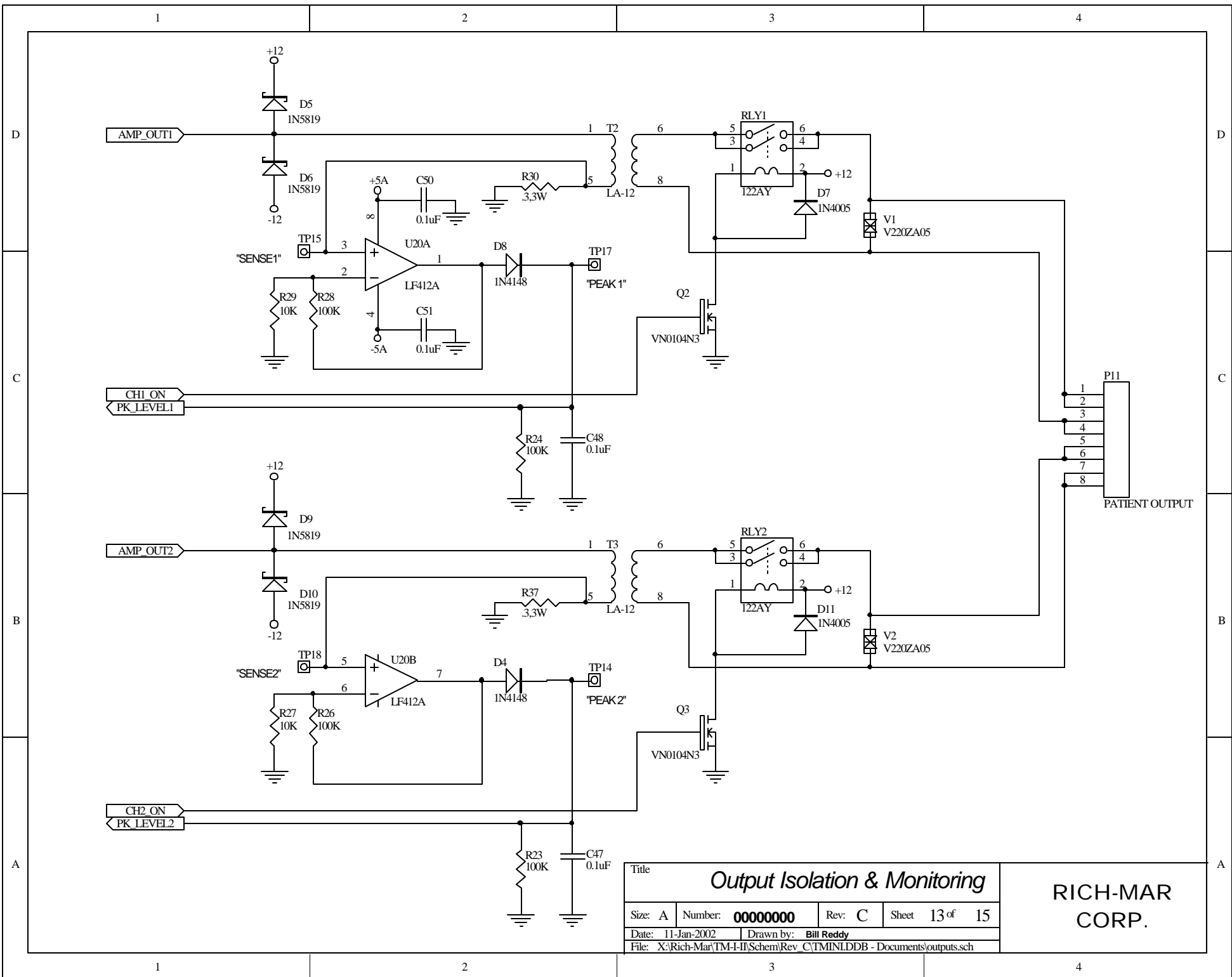

Title			
Output Isolation & Monitoring			
Size: A	Number: 00000000	Rev: C	Sheet 13 of 15
Date: 11-Jan-2002	Drawn by: Bill Reddy		
File: X:\Rich-Mar\TM-I-IT\Schem\Rev_C\TMINI.DDB - Documents\outputs.sch			

**RICH-MAR
CORP.**

1

2

3

4

D

D

C

C

B

B

A

A

Section III. POWER



- Power Supply
- Raw and regulated voltages

POWER SUPPLY
POWER.sch




Title				<i>Section III. POWER</i>		RICH-MAR CORP.	
Size: A	Number: 00000000	Rev: C	Sheet	14 of	15		
Date: 11-Jan-2002	Drawn by: Bill Reddy						
File: X:\Rich-Mar\TM-I-II\Schem\Rev_C\TMINLDDB - Documents\section3.sch							

1

2

3

4

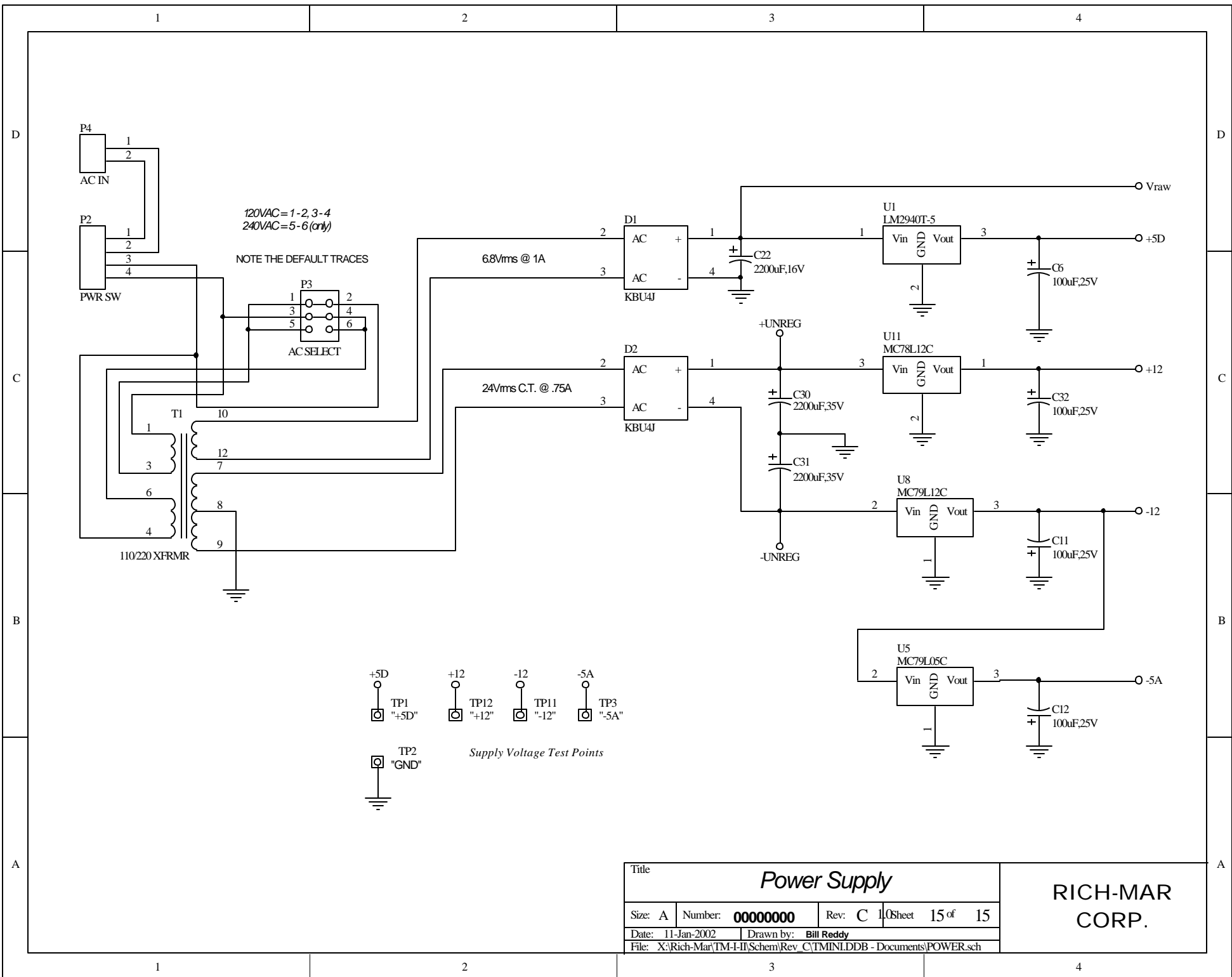

Title				Power Supply	
Size: A	Number: 00000000	Rev: C	1.0	Sheet 15 of 15	
Date: 11-Jan-2002	Drawn by: Bill Reddy				
File: X:\Rich-Mar\TM-I-IT\Schem\Rev_C\TMINI.DDB - Documents\POWER.sch					

**RICH-MAR
CORP.**