


## Calibration Instructions for the Intertron™ 6200

Soundhead S/N: \_\_\_\_\_

1. Attach the new soundhead to the Intertron™ 6200 and set it in the cradle.
2. Turn the unit on.
3. When the unit has finished the autotune sequence set the intensity to 1.0 W/cm<sup>2</sup>.
4. Select the ultrasound MENU window.
5. Select the CAL menu item.
6. When the CAL window is displayed select the POWER menu item.
7. When the POWER window is displayed enter the following key sequence:  
 SELECT  
 QUICKSET (on the stim side)  
 THERAPY  
 QUICKSET (on the stim side).  
 This will allow you to change the calibration values.
8. Select 1 MHz (indicated on the POWER window) by pressing the ultrasound QUICKSET key.
9. Use the touchpad up and down arrows to make the circled number match the 1 MHz calibration value listed below.  
  
 1 MHz Calibration Value: \_\_\_\_\_
10. Select 3 MHz by pressing the ultrasound QUICKSET key.
11. Use the touchpad up and down arrows to make the circled number match the 3 MHz calibration value listed below.  
 3 MHz Calibration Value: \_\_\_\_\_
12. Exit the POWER window and select the COUPLE menu item on the CAL window.
13. When the COUPLE window is displayed follow the instructions on the screen.
14. Exit the COUPLE window and set the soundhead in the cradle.
15. Press EXIT until the MAIN screen is displayed.
16. Calibration is complete.

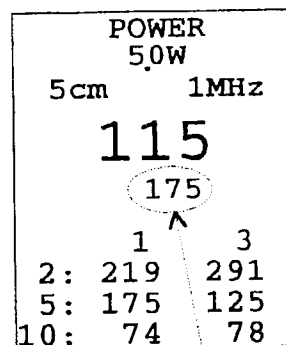

Calibration value

**CAUTION:** DO NOT GO BELOW THE FOLLOWING MINIMUM CALIBRATION FACTORS:

2CM <sup>2</sup> 1 MHZ <u>65</u>	3 MHZ <u>100</u>
5CM <sup>2</sup> 1 MHZ <u>78</u>	3 MHZ <u>84</u>
.10CM <sup>2</sup> 1 MHZ <u>72</u>	3 MHZ <u>51</u>