

Tim
Cell #
262-224-1236

1-262-224-1236

New from Williams Healthcare Systems

ZENITH III Verti-Lift Tables

1-800-406-7276

after this
weekend

Bob

Offer long term dependability
with enhanced features and benefits!

- Provides diagnostic circuitry to simplify (and using a matrix default, logically resolve) troubleshooting.
- Eliminates micro-switches and slide rails in favor of a clean and simple yo-yo potentiometer rated at millions of cycles for longer life and dependability.
- Lowers external circuitry to 5Volts

And best of all,

New -- Zenith III VertiLift circuit permits lift/lower travel up and down, '**JOG**' mode provides elevation without engaging the hylo direction.

- Same renowned structural framework patented for it's innovative locking design.
- Same model numbers and price point.
- Same colored base finishes available

Study this operator scenario:

Think of the Zenith III VertiLift as a table with two separate operative systems. One system operates the elevation cycle only, and the other traditional cycle operates hylo tilt/lift movements.

Here's a step by step description: Patient steps onto footplate. Doctor taps red timer/stop button: first (on Zenith II models this was done by kicking the Surestep) which engages a 20 second timer allowing the operator to run the table. Next tilt the table down by using the remote footswitch on the floor or the 'down' arrow button on the front end of the pipe