

frame. The table automatically travels to its destination. The Zenith III Verti-Lift Table will stop at the doctor's favorite height by the position of the new height position slide bar (side control penthouse) or travel can be interrupted by tapping either red stop button.

When finished, simply tap either red button again to initiate the timer circuit ('live' for twenty seconds) and hit either the raise side of the footswitch or down arrow button on end of the pipe.

Next are new user friendly features Zenith III VertiLift tables that you'll like a lot ...

While treating the patient it's possible to change the worktop height (lift or lower) for various techniques. Depress our new '**JOG**' button on the control penthouse (left side wing) and the table can be lowered or raised without tilting. In this cycle, Zenith III Verti-Lift tables operate in manual mode and will stop whenever the front arrow buttons or footswitch is released.

PS. All Zenith III tables are pre-wired for an optional second footswitch on the other side of the base, or the installer can simply move the standard provided footswitch to the rear if the customer prefers.

LOWER

Jog button

UP (FOOT SW)

UP OR DOWN (FOOT)

Red STOP

UP ON FOOT SW.