

## Calibrating Cox Table

- 1 Plug table into a power outlet.
- Note: Make sure the power is off before proceeding with step two.*



Table Must Be Plugged In

- 2 Push both the Start and Stop buttons at the same time. While holding these buttons in, push the Power button. Release Start and Stop buttons.
- Note: All three (3) lights will start flashing real fast.*



While Holding the Start And Stop Buttons Down Press the Power Button

Step	Actions	Picture
------	---------	---------

- 3 Push Start and Stop buttons at the same time while lights are flashing and hold until the Stop light stops flashing.
- NOTE: Stop light will go out, Start light will stay on. Rear Caudal Upper Section will move to the back of the table and then back to the front, at this point all lights will go off.*
- CALIBRATION IS COMPLETE.**



Press and Hold Start and Stop Buttons Until The Light Stops Flashing