



Excel Ultra Series Calibration Procedure.

1. *Turn on unit and allow it to boot and tune the applicator*
2. *Make a note of the date in the upper right corner of the screen*
3. *Press the assist button*
4. *Press the "Ultra Power Calibration button" (4th button down on right side of screen)*
5. *Enter the calibration "ID" number - (63 + the Day...i.e. 6320)*
6. *Hit enter button, next to "0" button on numeric keypad*
7. *Hit Continue*
8. *Allow the unit to tune with applicator in the applicator holder.*
9. *Load and align the applicator in the wattmeter*
10. *Hit Continue*
11. *Zero your wattmeter*
12. *The first setting is .5 watts, Hit the 2nd button down on the left side of screen*
13. *Enter the watts displayed on your wattmeter into the system and hit enter.*
14. *Hit the 2nd button down on the left side of screen and repeat entering the displayed value into the system for each setting.*
15. *Continue all 8 entries.*
16. *You can hit pause at anytime to re-zero your wattmeter.*
17. *Hit cancel if you make an entry mistake.*
18. *When complete, hit exit on the screen 3 times.*

**Call ERS Biomedical for more information or to order parts
800-479-2987**