



Operating Instructions for the Henley WEP style controller

Treatment Temperature: Rotate temperature control knob to desired setting.

Fluidization: Rotate blower control knob to desired setting

Starting Unit—Treatment time set: Depress “TMNT TIME” and “FAST” or “SLOW” button to set treatment time.

Heater light: Light is on when unit is heating.

To set time of day: Press TIME SET” and “FAST” or “SLOW” button am/pm light is on for pm

Programmed Preheat: (if desired) Press “PREHEAT” and “FAST” or “SLOW” button, until the display shows the time of day for the preheat to start. Preheat light is on when the preheat is activated. Allow 30-50 minutes for heat-up.

Cautions

Temperatures higher than 140 deg F may cause tissue damage.

Disconnect power supply before changing fuses.

Keep air inlets clean and free of obstructions.

Turn off unit while positioning patient.

Keep sleeves closed.